

1 Purpose

This document describes new functions, the changes, the improvements and fixed bugs in the following firmware version:

- 57.1008.0016 for OS version 006
- in comparison with the firmware version:
- 57.1008.0015 for OS version 006

2 New functions

The following new functions have been integrated:

- None

3 Improvements

The following improvements were made:

- The occurrence of the **Piston blocked** error message in SET titration was regulated.
- The creation of large data volume PDF reports has been improved.
- The stability of the remote control has been optimized.

4 Fixed bugs

The following bugs were fixed:

- DET mode hang-up (TITR-19660)
- The titration in SET mode stops before the endpoint is reached (TITR-19339).
- The buret is blocked (Error 010-005) during titration (TITR-19935).
- Sporadic freezing after pre-dosing (TITR-18647).

5 Known bugs

The following bugs are known:



- The PDF report also displays parameters that have not been activated (TITR-16729).
- Sporadic printing errors (TITR-17204)