

1 Purpose

This document describes new functions, changes, and improvements in the following firmware version:

- 57.1027.0011 for OS version 008

In comparison with the firmware version:

- 57.1027.0010 for OS version 008

2 New functions

The following new functions have been integrated:

- **Sample series** feature in all languages.

3 Improvements

The following improvements have been made:

- The method name is displayed on the **Live status** screen during conditioning/titration (except in Russian).

4 Fixed bugs

The following bugs have been fixed:

- After the statistics have been reset, the statistics counter is set to the next incremented value instead of taking the value from the changed method parameters.
- After switching to **DHCP** and then switching back to **static** again, the subnet mask shows a different value instead of the most recently set value.

5 Known issues

The following bugs are known:

- If the titration ends without dosed volume (EP volume = 0.0000 mL), then no chart will be printed.