

## 1 Purpose

This document describes the differences between the new software version **900 Touch Control 5.900.0010** and its predecessor PC Control 6.0 / Touch Control 5.840.0150.

## 2 General information



### Note

With this software release, PC Control gets discontinued. For service purposes, iService (based on PC Control) can be used.

## 3 New features

- Hot plug and play, USB devices can be connected and disconnected while the system is running
- Support of USB sticks (CF cards are not supported)
- Built-in Ethernet connection
- Saving and loading files through a Windows-share from a network computer
- Sending e-mails directly from Touch Control via mail server (SMTP)
- Printing to a network printer through printer share under Windows
- Native PDF generation
- New, user-friendly GUI design
- 6 predefined color schemes
- Electrode check (sensor connected or not)
- Manual titration command: methods for manual titration with (live) curve, calculation, reporting, data saving etc.
- Conditioning possibility under Manual control for double conditioning (e.g. 852 Titrand)
- Audit Trail viewer (separate application)

## 4 Improvements

- User name gets listed in result table report.

## 5 Fixed bugs and problems

- Routine dialog settings for favorites and custom electrode type are not stored.
- Text results do not get listed in the result table report.

## 6 Compliance

The current software version does not contain any modifications regarding 21 CFR Part 11.

**Herisau, 23 April, 2010**

---



D. Strohm

Vice President, Head of R&D